

2 in 1 Detergent and Disinfectant

for fixed surfaces and
non-critical articles.

Polymeric biguanide and
Quaternary of Ammonia
5th generation

Bombona Model 5L



Sprayer Model 750mL



READY FOR USE



QUICK ACTION



BIODEGRADABLE



ECONOMIC

PROVEN EFFICIENCY IN

2 min

- **Superfungus:**
 - Candida auris
- **Gram-negatives bacteria:**
 - Escherichia coli
 - Salmonella choleraesuis
 - Pseudomonas aeruginosa
- **Gram-positives bacteria:**
 - Enterococcus hirae
 - Staphylococcus aureus
 - Staphylococcus aureus MRSA
 - Enterococcus faecalis VRE



Characteristics:

- Neutral pH
- Alcohol free
- Polymeric Biguanide (2000ppm)
- Non-ionic surfactant
- 5th Generation Quaternary Ammonia (3000 ppm)
- No dyes
- No perfume
- non corrosive
- Non flammable

Atomic Ready is indicated for fixed surfaces and non-critical items:

Oximeter, Blood-pressure meter, Stethoscopes, Thermometers, Almotolias, urinals, buckets, basins, Diuresis Bottles, among others.



Oximeter



Stethoscope



Blood-Pressure Meter



Thermometer



Urinal



Almotolia

How to use:

To clean and disinfect fixed surfaces, apply Atomic Ready directly to the area using a spray bottle, sprayer or a clean cloth dampened with the product.

For cleaning and disinfection of non-critical articles, wash with Atomic Ready, leave to act for 2 minutes, if necessary, rinse and dry the articles.

While handling the product, we recommend the use of PPE.

