

# Biodegradable mild detergent for hospital use



Concentrated mild detergent with high cleaning power and excellent degreasing action, providing fast and effective cleaning in the removal of organic matter.



Dermatologically tested



Non-corrosive



Biodegradable



High yield



Ideal for cleaning fixed surfaces, instrumental surgical and dental-medical-hospital articles

# Mild detergent

For cleaning fixed surfaces,  
surgical instruments and  
dental-medical-hospital articles



- High cleaning power
- Degreasing action
- Non-ionic surfactant
- Non-corrosive



## Non-corrosive

Can be used with stainless steel, aluminum, surgical stainless steel, acrylic polypropylene, polyethylene, glass and other washable surfaces, not causing dryness on rubber surfaces.

## How to use:

Prepare the cleaning solution with Atomic DN-ULTRA, with water at room temperature, depending on the degree of soiling.

For heavy cleaning, dilute 250mL to 1 liter of water (250mL/L). For medium cleaning, dilute 20mL to 1 liter of water (20mL/L). for light cleaning, dilute 10mL to 1 liter of water (10mL/L).

The solution with Atomic DN-ULTRA can be applied directly to the surface, using a non-abrasive sponge or fiber. Leave to act for a few minutes and, if necessary, rub. When possible, leave items and utensils immersed in the diluted product solution. Rinse under running water.

