



Neutral biodegradable detergent for hospital use

Concentrated neutral detergent with high cleaning power and excellent degreasing action, providing fast and effective cleaning in removing organic matter.



Dermatologically tested



Non-corrosive



Biodegradable



High performance



Ideal for cleaning fixed surfaces, surgical instruments and dental-medical-hospital articles



Neutral Detergent

For cleaning fixed surfaces, surgical instruments and dental-medical-hospital articles

- High cleaning power
- Degreasing action
- Non-ionic surfactant
- Non-corrosive



Non-corrosive

It can be safely used on stainless steel, aluminum, surgical steel, acrylic, polypropylene, polyethylene, glass and other washable surfaces. It does not dry out rubber surfaces.

How to use

Prepare the cleaning solution with Atomic DN-ULTRA, with water at room temperature, depending on the level of dirt.

For heavy cleaning, dilute 250mL to 1 liter of water (250mL/L).
For medium cleaning, dilute 20mL to 1 liter of water (20mL/L).
For light cleaning, dilute 10mL to 1 liter of water (10mL/L).

The solution with Atomic DN-ULTRA can be applied directly to the surface, using a sponge or non-abrasive fiber. Leave to act for a few minutes and, if necessary, scrub. When possible, leave the items and utensils immersed in the diluted product solution. Rinse under running water.

